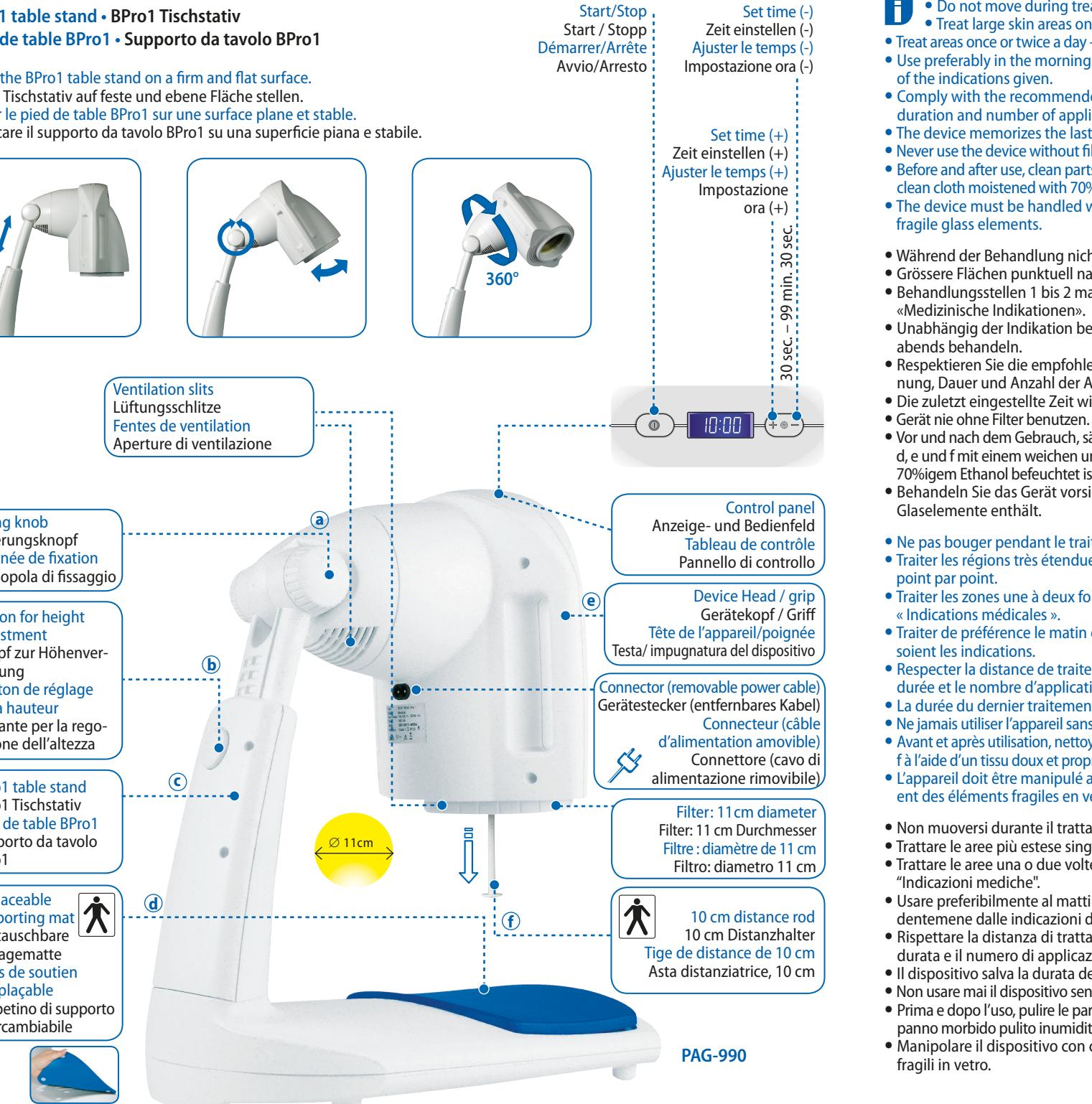
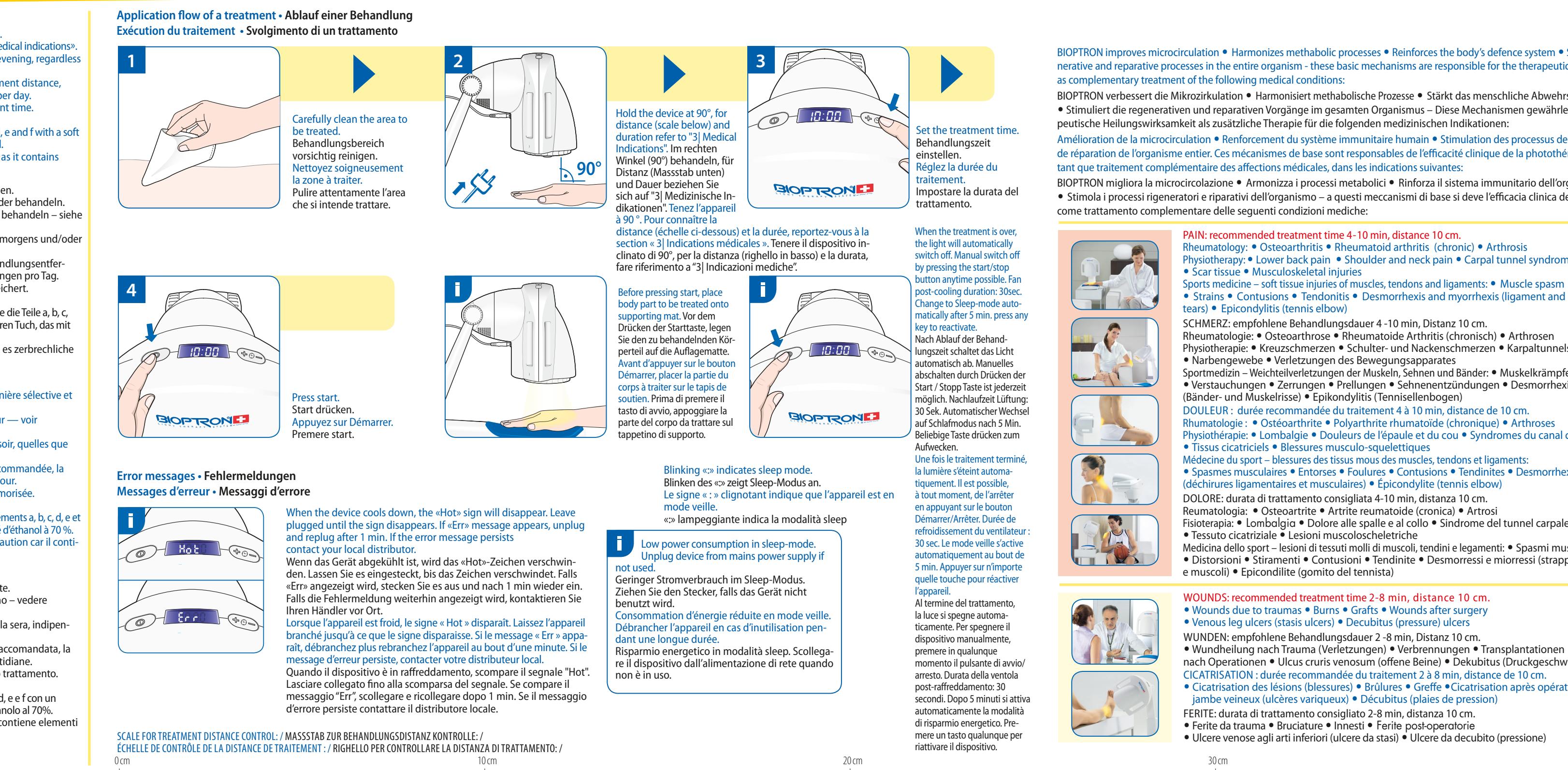
1 | Description of Device • Gerätbeschreibung Description de l'appareil • Descrizione del dispositivo

2 | BIOPTRON Light Therapy treatment • BIOPTRON Lichttherapie Behandlung • Traitement par photothérapie BIOPTRON • Trattamento fototerapico BIOPTRON

3 | Medical indications • Medizinische Indikationen • Indications médicales • Indicazioni mediche

DERMATOLOGICAL DISORDERS AND SKIN PROBLEMS: recommended treatment time 4-10 min, distance 10 cm.

- Atopic dermatitis • Psoriasis • Herpes simplex and zoster • Surface bacterial infections
- Mucosal lesions • Acne • Acne rosacea

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IN PEDIATRICS: APPLICATIONS ON CHILDREN UNDER 6 YEARS OLD MUST BE PERFORMED UNDER SUPERVISION OF A PHYSICIAN. Recommended treatment time 4-10 min, distance 10 cm.

- Pediatric dermal affections • Endogenous eczema • Upper respiratory tract infections • Allergic respiratory diseases • Pediatric musculoskeletal and neurological disorders and deficits

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PAIN: recommended treatment time 4-10 min, distance 10 cm.

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- Dermal affections as phlebitis, decubitus, intertrigo.

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This device may become a safety hazard if these instructions/safety precautions are ignored.

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Danger of electric shock! Keep dry! NEVER immerse in water! Do not open!

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Never block ventilation slits on the device.

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Never operate a damaged device (cables, plugs, housing or optical component).

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Risk of electric shock. Unplug the device if any liquids or particles enter the device. Contact your service center.

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Before use consult your doctor to see if this therapy is suitable for the intended treatment purpose.

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For the treatment of newborns use blindfold (PAG-500-BF).

Augenbinde für die Behandlung von Neugeborenen verwenden (PAG-500-BF). Pour le traitement des nouveau-nés, utiliser un bandeau (PAG-500-BF).

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Repairs and replacements of the halogen bulb should only be conducted by BIOPTRON-authorised service centres!

Do not carry out repairs yourself! No modification of this equipment is allowed! Danger if non-genuine substitute parts are used. The expected operational lifetime for the BPro1's device is 5 years and 10 years for the floor stand accessory. After that time, a maintenance service should be carried out.

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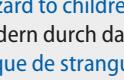
Do not look directly into light source. If not respected eye damage may occur. For the treatment of the eye area remove contact lenses and keep eyes closed.

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
The power cord poses a strangulation hazard to children.

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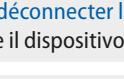
The power cord is used to disconnect the device from the mains power supply.

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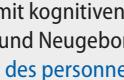
Applications on children and people with cognitive impairment always need to be performed under adult supervision.

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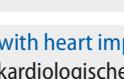BIOPTRON devices can be used by clients with heart implants and pacemakers.

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Contraindications and relative contraindications (*): before use consult a doctor, especially in the event of:

• Photo-allergy • Porphyria (all types) • Acute phototoxic (photo-allergic contact dermatitis) • Cheilitis exfoliativa actinica (Inflammation of lower lip due to radiation) • Erythema exudativum multiforme • Lupus erythematosus*

• Herpes solaris (Sun blisters) • Hydroa vacciniformis • Xeroderma pigmentosum (extreme sensitivity to UV rays)

• Eye inflammations* • Retinal diseases • Taking of photosensitizing pharmaceuticals/herbs.

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